6

Correct aligning

Incorrect aligning

7

✓ IDEAL
Upright

✓ GOOD
Slightly forward

✗ WON'T WORK
Bending forward

✗ MAY LEAK
Reclining back

✗ WON'T WORK
Laying back (or side)

8

A ✓
Correct aligning

B ✗
Incorrect aligning

9

Double Wearable In-Bra Breast Pumps MilkShell Model: 20/106

IMPORTANT SAFETY-RELATED COMMENTS

SAFETY-RELATED INFORMATION. Read **all instructions carefully** before using the BREAST PUMP. THE PRODUCT is intended for home use only. Always comply with basic safety precautions when using electrical appliances, especially when children are nearby. Read all instructions attached to this product before using it. Comply with the user instruction. Otherwise, personal injury or damage to the breast pump may occur.

WARNING

- Make sure you use a power supply that meets the breast pump specifications.
 - Keep the power supply away from sources of heat.
 - When disconnecting the power supply, first unplug it from the wall outlet and then from the breast pump.
 - Avoid using or storing the breast pump in places where it can easily fall down or into a bathtub or sink.
 - You can sterilize the breast pump components by boiling. The breast pump body is not heat resistant, so keep it away from heat sources and open flames.
 - Do not use the electric breast pump near flammable materials.
 - Do not leave the breast pump switched on unattended.
 - Do not use the breast pump if its power supply or cables are damaged. Do not use the breast pump if it has been dropped, damaged or is not working properly. If you find any signs of damage, stop using the breast pump immediately and get in touch with: service@canpolbabies.com.
 - Do not use the electric breast pump that has been in contact with water or other liquids. If the appliance has fallen into water or another liquid, do not touch it and immediately unplug the power supply from the socket.
 - Do not drench the breast pump or immerse it in water or other liquids.
 - Do not pour water over the breast pump.
 - Do not use the breast pump in the shower or bathtub.
 - Do not expose the breast pump to high temperatures.
 - Remember that a short circuit may occur if the USB Type-C port or the battery comes into contact with metal or liquid.
 - The product should not be serviced or repaired. Do not repair the appliance by yourself. Do not modify the appliance.
 - Do not use a damaged breast pump. Replace damaged or worn accessories with new ones immediately.
- Use the breast pump only for the purposes described in this user instruction.
 - Do not use the breast pump while sleeping or when you are very sleepy.
 - Do not use the breast pump while driving.
 - Check all parts of the breast pump before EACH use.
 - Expressing milk can cause uterine contractions. Do not express milk before giving birth. If you are pregnant and are still breastfeeding or expressing milk, consult your doctor before using the appliance again.
 - Do not cook or heat expressed milk in a microwave. Breast milk heated in a microwave becomes very hot and may cause serious burns to the baby's mouth. Heating in a microwave can also change the composition of the milk.
 - Use only parts recommended by Canpol sp. z o.o. with Canpol breast pumps.
 - Special care should be taken when using a breast pump or its parts near children.
 - The breast pump is not intended for use by people with physical, sensory or mental disabilities or by people who lack experience and knowledge, including children under 16. Children under 16 should use a breast pump only if they understand the risks involved and have received instructions on how to use the device safely or under adult supervision.
 - Keep all unused parts of the breast pump out of reach of children to avoid accidental swallowing or damage. Remember that this appliance and its parts are not toys.
 - The plastic container for milk and its components become fragile when frozen and can break due to a fall or impact.
 - Do not use milk from a damaged container or other damaged product for storage or feeding.
 - If you have problems with expressing milk or if expressing is painful, contact your lactation consultant. This product is for personal use only. Use by more than one person poses a risk for health.
 - Portable and radio communication devices can interfere with the operation of the breast pump.

USER INSTRUCTION

The MilkShell double portable shelly breast pump has been designed with lovers of the hands-free idea in mind. It is an ideal solution when you need to express larger amounts of milk every day, e.g. you are a KPI mother (mother breastfeeding differently) or you are returning to work. We hope that our breast pump will give you a lot of satisfaction and pleasure. Before using the product, read this user instruction carefully. Keep it for future reference, as it contains important information.

1. BATTERY MAINTENANCE

The breast pump contains a built-in (non-replaceable), lithium polymer battery that can be recharged.

WARNING

- If the battery is completely discharged, do not use the breast pump immediately after connecting it to the power supply. Leave the breast pump connected to the power supply for at least 20 minutes before you try to switch it on.
- To keep the breast pump battery in good technical condition, check that the power supply matches the voltage and current indicated in the product specification (5V 1A) before using it.

To maintain the correct life cycle of the battery:

- Store the breast pump in a cool place.
 - Recharge the battery before it becomes completely discharged. This is helpful to maintain the battery efficiency.
 - Fully charge the battery before storing the breast pump.
 - If the breast pump has been stored in a hot place, it should not be used at once. To restore the normal operation of the battery, wait an hour for the breast pump to cool down.
- During this time, the breast pump can be charged using the power supply plugged into the socket.

2. CHARGING-RELATED NOTES

NOTE

Power supply mode.

- Strictly follow the order of operations as indicated below.
- **Always use the DCSV 1A power supply. The power supply is not included in the set.**
- Make sure the power supply matches the voltage in the socket.

- Connect the USB Type-C cable of the power supply to the body of the appliance Fig. 1.
- Insert the power supply plug into the socket. Do not switch on the appliance body while connecting. Fig. 2

3. KIT COMPONENTS Fig. 3

- | | | |
|--|--|---|
| 1. Type C power cable (charger not included in the set), | 8. Silicone tube 2 pcs., | To learn how to choose the right size of the funnel, go to: |
| 2. Breast pump control device, | 9. Silicone funnels (3 different sizes: 19mm, 21 mm & 24 mm) 6 pcs., | |
| 3. Silicone membrane 2 pcs., | 10. Breast pump hanging strap lanyard 2 pcs. | |
| 4. Non-return valve 2 pcs., | 11. Extension for a bra shoulder strap 2 pcs. | |
| 5. Breast funnel 27 mm pcs., | | |
| 6. Milk collection shell/container 2 pcs., | | |
| 7. Membrane closure/cover 2 pcs., | | |

4. OPERATING THE BODY

Description of functions on the control panel Fig. 4

- A – on/off/pause,
- B – decreasing suction power,
- C – increasing suction power,
- D – mode switch,
- E – alternating breast stimulation mode,
- F – alternating milk expressing mode,

- G – simultaneous breast stimulation mode,
- H – simultaneous milk expressing mode,
- I – battery level indicator,
- J – time indicator,
- K – display,
- L – charging socket,
- M – input sockets for silicone tubes.

E – Alternating breast stimulation mode – stimulation and preparation of breasts for milk production and outflow. Stimulation is done alternately, i.e. once from one breast, and once from the other. This is a fast mode, more intensive than the other modes due to more cycles per minute. Mode with an option of adjusting the intensity. From the stimulation mode, the mode should be changed after 2-3 minutes, unless we notice the earlier milk outflow.

F – Alternating milk expressing mode – proper milk expressing. The milk expressing mode is performed alternately, i.e. once from one breast, and once from the other with an option of adjusting the intensity.

If you want to use the breast pump only for one breast, then close the opening of the input socket on the opposite side of appliance using the plug that is an integral part of the appliance.

G – Simultaneous breast stimulation mode – stimulation and preparation of the breasts for milk production and outflow. Stimulation is done at the same time for both breasts. Mode with an option of adjusting the intensity. This mode is with less number of cycles per minute due to the fact that the action for both breasts takes place at the same time.

H – Simultaneous milk expressing mode – proper milk expressing from both breasts at the same time. Less intensive mode with an option of adjusting the intensity.

The breast pump remembers the last used operating mode, so you can easily use the same settings the next time you express the milk.

5. CLEANING AND DISINFECTION

Keep the product in cleanliness.

Charging cable and breast pump body:

- Neither wash nor disinfect in water.
- Wipe the surface with a paper towel.
- If the appliance is accidentally dropped into water or other liquid residues, discontinue use and contact customer service.

Other parts of the breast pump:

- Can be washed by hand, with a mild detergent. Before first use and after each subsequent use, wash the parts under running water with a mild detergent, rinse thoroughly with warm water. Pour boiling water over it.
- Can be disinfected with steam and boiling water. After boiling the water, switch off the heat source and wait three minutes before disinfecting.
- Leave the parts to dry.
- Store all components in a ventilated room.

6. ASSEMBLY

Assemble the breast pump components Fig. 5.

Disassemble and clean the parts that come into contact with the breast and breast milk immediately after use to avoid bacteria from breast milk residues.

1. Assemble the valve membrane on the funnel. Fig. 5.1
2. Assemble the non-return valve under the funnel. Fig. 5.2
3. Assemble the milk collection container as shown in the drawing. Fig. 5.3
4. Press firmly on both sides with both hands to make sure that the whole fits together. Fig. 5.4
5. Assemble the closure on the membrane and press it firmly to prevent it from falling off. Fig. 5.5
6. Use silicone tubes to connect the appliance Fig. 5.6

All components must be properly assembled, otherwise it will affect the operation of the appliance and milk may leak out.

Assembly tips

WARNING

- Before touching any components of the breast pump or breast, wash your hands thoroughly with soap and water (for at least one minute) and dry them with a clean towel or disposable paper towel.
- Remember that milk containers and their components can be damaged by improper use (e.g. due to falling, overtightening or tipping over).
- Care should be taken when using milk containers and their components.
- Use only original, electrical parts of the breast pump. Using substitutes can cause irreversible damage.
- Before assembly, make sure that all components are completely dry.
- Assemble all components with care, step by step, according to the instructions. Otherwise, the breast pump may not generate the appropriate vacuum.

7. USE

- Before milk expressing, it is recommended to wash your hands and dry them with a paper towel.
- Short press the power supply button/switch to switch on the appliance.
- When the appliance is on, short press the power supply button to stop its operation.
- Another short press of the power supply button to start again. Press and hold the power supply button to switch off the **appliance**.
- NOTE: If the power supply is switched off due to insufficient power, it can be switched on again after charging.
- Each mode has 1-12 levels. Press **plus** - level increases or **minus** - level decreases to adjust the force value.
- Short press the M button to switch to different modes.
- Each time you press the power supply button to switch it on, the appliance will first enter the stimulation mode, and after two minutes of stimulation, it will automatically enter the milk expressing mode.
- During use, the force value can be adjusted by pressing **plus** - level increase or **minus** - level decrease.

The appliance contains a 1200mAh lithium battery, which can store electricity and collect milk. One full charge takes about 3 hours. When the battery capacity falls below 30%, I - (battery level LED indicator) will display a flashing light, you should start charging. When charging, I - (battery LED indicator) will display a flashing light. When it is full, the full charge indicator will be on.

8. PROPERLY PUTTING ON THE SHELL

Wash your hands thoroughly. Before use, remember that skin on the breast and the funnel of the shell must be dry. Insert the assembled funnel with the milk container into your bra so that it has proper position and that your nipple is inside the nipple tunnel and your mammary papilla is sticking out through the round hole. SEE FIG. 6

Place comfortably the assembled milk containers in your bra. When expressing the milk, remember not to tilt your chest back and forth, as this can cause milk to leak out see Fig. 7.

RECOMMENDATION: When expressing the milk, try to keep the upper part of your body straight. Lean forward or back by a maximum of 45°. IF YOU LEAN TOO MUCH OR HAVE A LOT OF MILK EXPRESSED (OVER 100 ML) AND YOU ARE IN MOTION, IT MAY ESCAPE FROM THE CONTAINER THROUGH THE VENT HOLE TO THE OUTSIDE. WHEN YOU NOTICE MILK DRIPPING OUT FROM UNDER THE BREAST PUMP, EMPTY THE CONTAINER AND CONTINUE MILK EXPRESSING.

The silicone tube should be outside the bra, hanging freely. Make sure it is not twisted or folded. This will allow the milk to flow freely into the container.

SEE FIG. 8a - **correct**, 8b - **incorrect**.

Note: Wearing a nursing bra or breast pump for a long time can cause discomfort in the mammary glands. Remove the containers when you feel pain or discomfort. Control how often and for how long you use the breast pump. When you finish milk expressing, unfasten the nursing bra and slowly remove the breast pump.

9. POURING MILK

When removing the breast pump, do not turn it upside down, but prepare a bottle for storing milk and pour out the milk as per the correct method of pouring milk shown in Fig. 9. After use, wash all parts of the breast pump as per the cleaning instructions.

10. STORING MILK

- Due to the active biochemical and microbiological processes occurring in milk, try to store milk at a constant temperature.
- Expressed milk can be stored in bottles, sterile bags or airtight containers with a lid (which should be disinfected before each use).
- Mark each container of expressed milk with a date.
- Freeze only freshly expressed milk. Do not fill the container to the brim, as frozen milk expands its volume.
- Expressed and frozen milk can be thawed in the same way as other foodstuffs, except in a microwave oven. (Using a microwave oven can deprive the milk of valuable nutrients and antibodies).
- Frozen milk can be thawed overnight in the refrigerator and then warmed gradually in a milk warmer or in hot water.
- Shake or stir the milk after thawing to bind the separated fat molecules with the remaining milk volume.
- DO NOT store milk on the shelves located on the refrigerator door.
- NEVER freeze again previously thawed milk.
- DO NOT mix freshly expressed milk with thawed milk.
- DO NOT mix freshly expressed milk with previously frozen milk.
- NEVER heat the expressed milk in microwave or boil it.
- ALWAYS check the temperature of the food before serving.
- Throw away any unused milk.

Type of breast milk	Temperature (°C)	Recommended time of storage
Fresh milk	+4°C	up to 96 hours
	from +4°C to +15°C	maximum 24 hours
	+19°C to +25°C (room temperature)	optimally up to 4 hours
Frozen milk	from +25°C to +37°C	maximum up to 4 hours
	-20°C	optimally 3-6 months
Thawed milk	+4°C	up to 24 hours (since total thaw)
	from +25°C to +37°C	up to 4 hours

11. TECHNICAL PARAMETERS

Model: 20/106
 Weight: 170g
 Charger parameters: 110-240V, 50-60 Hz / output 5V / 1A
 Built-in 3.7V 1200mAh battery
 Generated negative pressure: 280mmhg ± 20%
 Charging time: about 2h
 Work time: 1.5h, 3 cycles

If you have any doubts about the safety of the product, please contact with distributor or manufacturer. The information contained in this manual is subject to appropriate technical changes.

This marking indicates that the product should not be disposed of with other household wastes. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of waste and recycle it responsibly to promote the sustainable reuse of material resources. For further information on the recycling of this product, please contact the municipal authorities in their area of waste disposal service or the supplier of the product. This product and its information meet the requirements of the WEEE Directive 2012/19/EU.

The manufacturer declares that the product complies with European Union directives and harmonized standards. The product has been marked by conformity assessment procedures.

The product meets the requirements of the directive concerning electrical equipment designed for use within certain voltage limits for power adapter (LVD) 2014/35/EU, the Electromagnetic Compatibility Directive (EMC) 2014/30/EU, and RoHS Directive 2011/65/EU.

It is the Class III appliance

This product is not a toy. Keep all parts away from children. Keep the instructions of use as it contains important information. Batch code is placed on the packaging.

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Manufacturer: **Canpol Sp. z o.o.**

Puławska Street 430, 02-884 Warsaw, Poland

Object of the declaration: **Double Wearable In-Bra Breast Pumps MilkShell**

Brand: **Canpol Babies**

Model / cat. no.: **20/106**

In relation to the described product, the manufacturer declare under the sole responsibility that the product to which this declaration refers is in line with the following European Union directives:

2014/30/EU Electromagnetic compatibility (EMC)

2011/65/EU Restrictions of the use of certain hazardous substances in electrical and electronic equipment (RoHS)

The product is in line with harmonized standards and other technical standards:

- EN 60335-1:2012 + AC:2014 + A1:2019 + A2:2019 + A11:2014 + A13:2017 + A14:2019 + A15:2021 + AC:2014
- EN 60335-2-15:2016 + A1:2021 + A2:2021 + A11:2018 + A12:2021
- EN 55014-1:2017 + A1:2020
- EN 55014-2:1997 + A1:2001 + A2:2008 + AC:1997 [EN 55014-2:2019]
- EN 61000-3-2:2014 (EN IEC 61000-3-2:2019)
- EN 61000-3-3:2013 (+ A1:2017 + A2:2021)
- EN IEC 63000:2018

The last two digits of the year in which the marking was applied

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Adrianna Bogała
Research and Documentation Specialist

Warsaw, 23.10.2024

(Place and date of issuing the declaration)

(name, surname, position and signature of the authorized person)

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