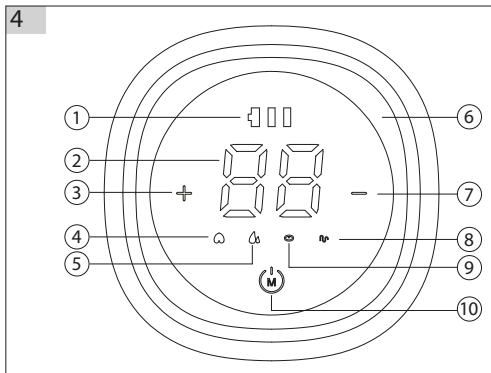
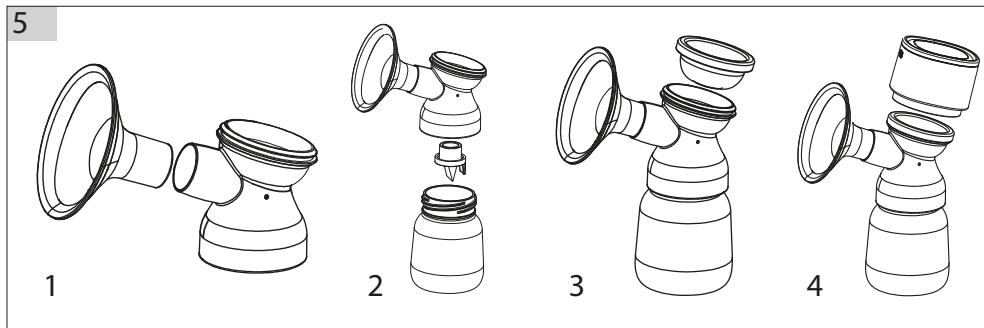
4

5

Portable Electric Breast Pump PowerPump Model: 20/107

SAFETY-RELATED INFORMATION. Read **all instructions carefully** before using the BREAST PUMP. THE PRODUCT is intended for home use only.

IMPORTANT SAFETY-RELATED COMMENTS

Always comply with basic safety precautions when using electrical appliances, especially when children are nearby. Read all instructions attached to this product before using it. Comply with the user instruction. Otherwise, personal injury or damage to the breast pump may occur.

WARNING

- Make sure you use a power supply that meets the breast pump specifications.
- Keep the power supply away from sources of heat.
- When disconnecting the power supply, first unplug it from the wall outlet and then from the breast pump.
- Avoid using or storing the breast pump in places where it can easily fall down or into a bathtub or sink.
- You can sterilize the breast pump components by boiling. The breast pump body is not heat resistant, so keep it away from heat sources and open flames.
- Do not use the electric breast pump near flammable materials.
- Do not leave the appliance switched on unattended.
- Do not use the breast pump if its power supply or cables are damaged. Do not use the breast pump if it has been dropped, damaged or is not working properly. If you find any signs of damage, stop using the breast pump immediately and get in touch with: serwis@canpolbabies.com.
- Do not use the electric breast pump that has been in contact with water or other liquids. If the appliance has fallen into water or another liquid, do not touch it and immediately unplug the power supply from the socket.
- Do not drench the breast pump or immerse it in water or other liquids.
- Do not pour water over the breast pump.
- Do not use the breast pump in the shower or bathtub.
- Do not expose the breast pump to high temperatures.
- Remember that a short circuit may occur if the USB Type-C port or the battery comes into contact with metal or liquid.
- The product should not be serviced or repaired. Do not repair the appliance by yourself. Do not modify the appliance.
- Do not use a damaged breast pump. Replace damaged or worn accessories with new ones immediately.

- Use the breast pump only for the purposes described in this user instruction.
 - Do not use the breast pump while sleeping or when you are very sleepy.
 - Do not use the breast pump while driving.
 - Check all parts of the breast pump before EACH use.
 - Expressing milk can cause uterine contractions. Do not express milk before giving birth.
- If you are pregnant and are still breastfeeding or expressing milk, consult your doctor before using the appliance again.
- Do not cook or heat expressed milk in a microwave. Breast milk heated in a microwave becomes very hot and may cause serious burns to the baby's mouth. Heating in a microwave can also change the composition of the milk.
 - Use only parts recommended by Canpol sp. z o.o. with Canpol breast pumps.
 - Special care should be taken when using the breast pump or its parts near children.
 - The breast pump is not intended for use by people with physical, sensory or mental disabilities or by people who lack experience and knowledge, including children under 16. Children under 16 should use a breast pump only if they understand the risks involved and have received instructions on how to use the device safely or under adult supervision.
 - Keep all unused parts of the breast pump out of reach of children to avoid accidental swallowing or damage. Remember that this appliance and its parts are not toys.
 - The plastic container for milk and its components become fragile when frozen and can break due to a fall or impact.
 - Do not use milk from a damaged container or other damaged product for storage or feeding.
 - If you have problems with expressing milk or if expressing is painful, contact your lactation consultant. This product is for personal use only. Use by more than one person poses a risk for health.
 - Portable and radio communication devices can interfere with the operation of the breast pump.

USER INSTRUCTION

The portable Canpol babies PowerPump electric breast pump will be ideal for stimulating lactation in the first days after the delivery, when having difficulties in initiating lactation, e.g. after a caesarean section or milk deficiencies. Regular milk expressing from the very beginning can help stabilizing and increasing milk production. The breast pump with a built-in control panel is simple and convenient to operate. We hope that our breast pump will give you a lot of satisfaction and pleasure. Before using the product, read this user instruction carefully. Keep it for future reference, as it contains important information.

1. BATTERY MAINTENANCE

The breast pump contains a built-in (non-replaceable), lithium polymer battery that can be recharged.

WARNING

- If the battery is completely discharged, do not use the breast pump immediately after connecting it to the power supply. Leave the breast pump connected to the power supply for at least 20 minutes before you try to switch it on.
- To keep the breast pump battery in good technical condition, check that the power supply matches the voltage and current indicated in the product specification before using it. To maintain the correct life cycle of the battery:
- Store the breast pump in a cool place.

- Recharge the battery before it becomes completely discharged. This is helpful to maintain the battery efficiency.
- Fully charge the battery before storing the breast pump.
- If the battery indicator flashes during use or flashes when the appliance is switched on, it means that the battery is weak. Then it needs to be charged.
- The battery indicator will flash while charging. It will stop flashing when the battery is fully charged.
- If the breast pump has been stored in a hot place, it should not be used at once. To restore the normal operation of the battery, wait an hour for the breast pump to cool down. During this time, the breast pump can be charged using the power supply plugged into the socket.

2. CHARGING-RELATED NOTES

NOTE

Power supply mode.

- Strictly follow the order of operations as indicated below.
- **Always use the DCSV, 1A power supply. The power supply is not included in the set.**

- Make sure the power supply matches the voltage in the socket.
- Connect the USB Type-C cable of the power supply to the body of the appliance Fig. 1.
- Insert the power supply plug into the socket. Do not switch on the appliance body while connecting. Fig. 2

3. KIT COMPONENTS Fig. 3

- | | |
|---|---|
| <ol style="list-style-type: none"> 1. Control device, 2. Silicone membrane, 3. Breast pump body, 4. Non-return valve, | <ol style="list-style-type: none"> 5. Milk container, 6. Funnel 22mm, 7. Type C power cable (charger not included in the set). |
|---|---|

4. CONTROL DEVICE OPERATION

Description of functions on the control panel Fig. 4

- | | |
|---|---|
| <ol style="list-style-type: none"> 1. battery level indicator 2. time and mode indicator 3. level increase button 4. breast stimulation mode 5. milk expressing mode | <ol style="list-style-type: none"> 6. display 7. level decrease button 8. mix mode 9. lactation mode 10. on/off switch |
|---|---|

- 4 - breast stimulation mode - preparation of breast for milk outflow, stimulation of lactation,
- 5 - proper milk expressing mode,
- 9 - lactation mode - consists of 5 stimulation cycles and 1 milk expressing cycle,
- 8 - mix mode - consists of 3 stimulation cycles, 2 massage cycles and then 2 stimulation cycles and 1 massage cycle.

If you want to switch modes, briefly press the „M“ button. If you want to switch it off, hold the button for 2 seconds. Press and hold again for 2 seconds to switch it off. The product switches off automatically after about 30 minutes of continuous operation.

LAMP

The device has a lamp. To turn the light on/off, hold the minus, or plus and minus buttons simultaneously.

You can set 1 to 15 power levels for each mode, using the „plus“ and „minus“ buttons.

5. CLEANING AND DISINFECTION

- Keep the product in cleanliness.
- Before first use and before each subsequent use, wash the membrane, body, valve and milk container. You can boil them, by immersing for 5 minutes in boiling water and disinfect in steam sterilizer. Do not disinfect the funnel in such a way. Before first use and after each subsequent use, wash the funnel under running water with a mild detergent, rinse thoroughly with warm water. Pour boiling water over it.
- Do not wash or disinfect the control device in a steam sterilizer, it can simply be wiped with a soft or a slightly damp, tightly wrung cloth.
- **WARNING!** When cleaning, handle the non-return valve and silicone membrane with care, if they are damaged, the breast pump will not function properly.
- To clean the non-return valve, rub it gently, without inserting any objects, as this may result in damage.
- Before use, make sure there is no any remaining water in the silicone membrane to prevent damage to the control device.
- Store all components in a ventilated room.
- If you have any doubts as to the safety of the product, please contact the distributor or the manufacturer.

6. ASSEMBLY

Assembly tips:

WARNING

- Before touching any breast pump components or breast, wash your hands thoroughly with soap and water (for at least one minute) and dry them with a clean towel or disposable paper towel.
- Remember that milk containers and their components can be damaged by improper use (e.g. due to falling, overtightening or tipping over).
- Care should be taken when using milk containers and their components.
- Use only original, electrical parts of the breast pump. Using substitutes can cause irreversible damage.
- Before assembly, make sure that all components are completely dry.
- Assemble all components with care, step by step, according to the instructions. Otherwise, the breast pump may not generate the appropriate vacuum.
- When using the device for the first time, it must be fully charged, at the first place.
- Connect the breast pump body to the funnel. Fig. 5.1
- Place the non-return valve on the bottom of the body. Push it all the way in and make sure it is tightly fitted. Then screw the milk container into the body opening and tighten. Fig. 5.2
 - Mount the silicone membrane on the body from above so that the groove's edge covers the round edge of the body and ensures a tight fit. Fig. 5.3
- Press the control device onto the mounted membrane until you feel resistance. Press it into place. Fig. 5.4

7. USE

- Wash your hands thoroughly before use. Make sure that all breast pump components have been thoroughly cleaned and check again that all parts are properly mounted.
- Before breastfeeding, putting on a warm towel and massaging the breasts can enhance the effect of milk expressing. Sit comfortably and relax, placing a footrest under your feet to raise your legs, which helps maintaining proper posture and prevent pressure on your back.
- Before use, remember that skin on the breast and the funnel must be dry.
- Place the assembled funnel pressing it firmly against your breast while supporting your breast with your other hand to ensure tightness.
- Press the power supply button to switch on the appliance, which will automatically switch to stimulation mode when switched on. You can now select the mode of your choice.
- While the appliance is operating, pressing the force increase or decrease button allows selecting from 1 to 15 intensity levels of the selected mode.
- To select or adjust the power level that is right for you in the selected mode, gradually increase the power until you feel slight discomfort (not pain), then reduce the power slightly.
- After milk expressing, press and hold the power supply button for 2 seconds to switch the appliance off.
- The appliance does not remember your last settings and does not automatically switch to another mode.

After use wash all breast pump components as per the cleaning instructions.

8. MILK STORAGE

- Due to the active biochemical and microbiological processes occurring in milk, try to store milk at a constant temperature.
- Expressed milk can be stored in bottles, sterile bags or airtight containers with a lid (which should be disinfected before each use).
- Mark each container of expressed milk with a date.
- Freeze only freshly expressed milk. Do not fill the container to the brim, as frozen milk expands its volume.
- Expressed and frozen milk can be thawed in the same way as other foodstuffs, except in a microwave oven. (Using a microwave oven can deprive the milk of valuable nutrients and antibodies).
- Frozen milk can be thawed overnight in the refrigerator and then warmed gradually in a milk warmer or in hot water.
- Shake or stir the milk after thawing to bind the separated fat molecules with the remaining milk volume.
- DO NOT store milk on the shelves located on the refrigerator door.
- NEVER freeze again previously thawed milk.
- DO NOT mix freshly expressed milk with thawed milk.
- DO NOT mix freshly expressed milk with previously frozen milk.
- NEVER heat the expressed milk in microwave or boil it.
- ALWAYS check the temperature of the food before serving.
- Throw away any unused milk.

Milk type	Temperature (°C)	Recommended storing time
Fresh milk	+4°C	up to 96 hours
	from +4°C to +15°C	maximum up to 24 hours
	+19°C to +25°C (room temperature)	optimally up to 4 hours
Frozen milk	from +25°C to +37°C	maximum up to 4 hours
Thawed milk	-20°C	optimally up to 3-6 months
	+4°C	up to 24 hours (since total thaw)
	from +25°C to +37°C	up to 4 hours

9. TECHNICAL PARAMETERS

Model: 20/107
 Weight: 237g
 Charger parameters: 110-240V, 50-60 Hz / output 5V / 1A
 Built-in 3.7V 1200mAh battery
 Generated negative pressure 42 ± 3 kpa (360 mmHg)
 Charging time: about 2h
 Work time: 2h, 4 cycles

If you have any doubts about the safety of the product, please contact with distributor or manufacturer. The information contained in this manual is subject to appropriate technical changes.

This marking indicates that the product should not be disposed of with other household wastes. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of waste and recycle it responsibly to promote the sustainable reuse of material resources. For further information on the recycling of this product, please contact the municipal authorities in their area of waste disposal service or the supplier of the product. This product and its information meet the requirements of the WEEE Directive 2012/19/EU.

The manufacturer declares that the product complies with European Union directives and harmonized standards. The product has been marked by conformity assessment procedures.

The product meets the requirements of the directive concerning electrical equipment designed for use within certain voltage limits for power adapter (LVD) 2014/35/EU, the Electromagnetic Compatibility Directive (EMC) 2014/30/EU, and RoHS Directive 2011/65/EU.

It is the Class III appliance

This product is not a toy. Keep all parts away from children. Keep the instructions of use as it contains important information. Batch code is placed on the packaging.

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Manufacturer: **Canpol Sp. z o.o.**

Pulawska Street 430, 02-884 Warsaw, Poland

Object of the declaration: **Portable Electric Breast Pump Power Pump**

Brand: **Canpol Babies**

Model / cat. no.: **20/107**

In relation to the described product, the manufacturer declare under the sole responsibility that the product to which this declaration refers is in line with the following European Union directives:

2014/30/EU Electromagnetic compatibility (EMC)

2011/65/EU Restrictions of the use of certain hazardous substances in electrical and electronic equipment (RoHS)

The product is in line with harmonized standards and other technical standards:

- EN 60335-1:2012 + AC:2014 + A1:2019 + A2:2019 + A11:2014 + A13:2017 + A14:2019 + A15:2021 + AC:2014
- EN 60335-2-15:2016 + A1:2021 + A2:2021 + A11:2018 + A12:2021
- EN 55014-1:2017 + A11:2020
- EN 55014-2:1997 + A1:2001 + A2:2008 + AC:1997 (EN 55014-2:2019)
- EN 61000-3-2:2014 (EN IEC 61000-3-2:2019)
- EN 61000-3-3:2013 (+ A1:2017 + A2:2021)
- EN IEC 63000:2018

The last two digits of the year in which the marking was applied

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Adrianna Bępała
Research and Documentation Specialist

Warsaw, 23.10.2024

(Place and date of issuing the declaration)

(name, surname, position and signature of the authorized person)

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Adeline Bogucka
Research and Documentation Specialist

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