


Pumping doesn't have to be painful

How to choose the right size of the funnel?

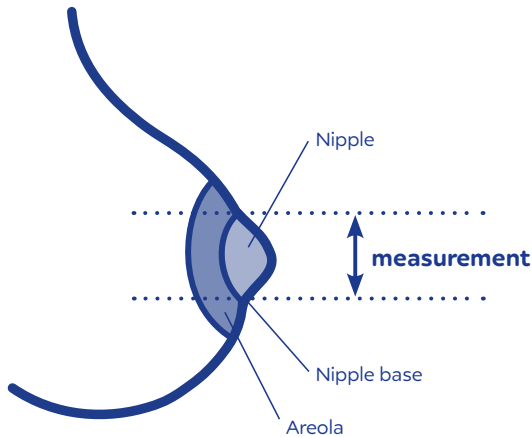

The funnel has a significant impact on the comfort and effectiveness of milk expression. Choosing the right funnel size makes it easier to achieve optimum milk flow.

The size of the breast does not necessarily correspond to the size of the nipple. The diameter of the nipple determines the size of the funnel.

Using a tape measure or ruler, measure the diameter of the nipple at its base (through the centre), not including the areola. Read the size in millimetres (mm).

REMEMBER!


Sometimes it is necessary to choose a different breast shield size for each breast – measure the nipple in both breasts.

Double Electric Breast Pump ExpressCare & Mobile Electric Breast Pump Take&Go & Electric Breast Pump EasyStart



Try the funnels included with Canpol babies breast pump in the set, both with and without a silicone breast shield.


1. Gently place the funnel with the silicone cover on your breast with the nipple in the center.
2. Set the maximum suction power that is comfortable for you. Pumping shouldn't be painful.
3. During the actual pumping phase, after the stimulation phase, check that you have the correct size funnel, using the following graphics as a guide.

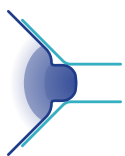

If the nipple rubs against the walls of the tunnel – **the funnel is too small**. Remove the silicone and try again. If the situation repeats, try larger funnel size.



When the nipple is in the middle of the tunnel and moves freely means that the **funnel is correct**.

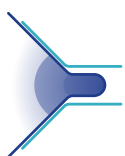

When the nipple is retracted into the funnel tunnel with the part of the areola – **funnel is too big**, try a smaller size.

Breast pump funnel with flexible rim 27 mm*



*Not part of the set, available for purchase on canpolbabies.com website.