GB • A washing bottle with a delicate, refreshing and revitalizing water stream makes it easier to care for the perineum while using the toilet, thus reducing the risk of possible infections or additional irritations. The bottle is great as additional support in maintaining daily hygiene, during pregnancy and after childbirth, during menstruation, trips or infections. Wash the bottle and its parts in warm soapy water, periodically pour with boiling water. Batch code is placed on the packaging. Retain the instruction for later use.

How to use the Canpol babies postpartum bottle?

1. While on the toilet, pour warm water into the bottle and screw the cap tightly on. The bottle has 2 tips with different jet angles for convenient use.

2. Turn the bottle upside down, point it between your legs, keeping the nozzle pointing towards your perineum, and gently squeeze the bottle.

3. You can use the bottle while using the toilet, so you won't feel any stinging or burning, or just to clean your intimate areas.

4. After use, unscrow, rinse and dry the bottle. Attention! The bottle is not waterproof – its contents may spill during transport. Before you throw it into the bag, empty it, dry and put it in the bag attached to the product.

Attention! The bottle is not for irrigation. We only wash the external parts of intimate places.

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